## CS2 Small Cuff

Small cuff

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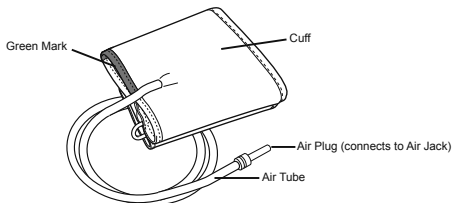
Please follow this instruction manual thoroughly for your safety.


Please keep for future reference. For specific information about your own blood pressure, CONSULT YOUR PHYSICIAN.

### Instructions for use
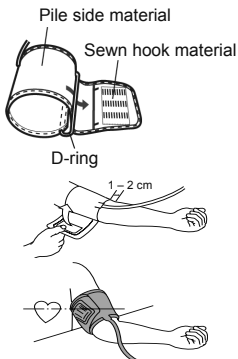
#### Instruction Manual for Small Cuff

Thank you for purchasing the OMRON Small Cuff. Arm circumference for this cuff is 17 to 22 centimeters. OMRON Small Cuff can be connected to various OMRON Blood Pressure Monitors. Please refer to the original instruction manual of your Blood Pressure Monitor.

Please check your present product as to which plug you must use. Using the wrong plug may cause air leakage and incorrect readings.


#### Applying the Arm Cuff

1. Put your left arm through the cuff loop.

**Note:** If the cuff is not assembled, pass the end of the cuff furthest from the tubing through the metal D-ring to form a loop. The smooth cloth should be on the inside of the cuff loop.

2. Position the arm correctly.

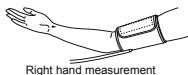- 1) The air tube should run down the inside of your forearm and be in line with your middle finger.
- 2) The bottom of the cuff should be approximately 1 to 2 cm above your elbow.
- 3) Apply the cuff to your upper arm so that the marker (arrow under tube) is centered on the middle of your inner arm and points down the inside of the arm.

## Hints

## Maintenance

### Taking measurements on the right arm

Apply the cuff so that the air tube is at the side of your elbow.

- Be careful not to rest your arm on the air tube, or otherwise restrict the flow of air to the cuff.
- The cuff should be 1 to 2 cm above the elbow.

3. When the cuff is positioned correctly, close the fabric fastener **FIRMLY**.

- Do not place the cuff over thick clothes and do not roll up your sleeve if it is too tight.
- Align the cuff with the height of your heart (breast height). If necessary use a pillow or cushion.

- Stains on the cuff can be carefully removed using a moist cloth and soapy water.

**Do not wash the cuff or make it wet!**

- Do not fold the cuff or air tube together too tightly and do not crease them.
- Do not store the cuff in the following situations:
  - If the cuff is wet.
  - Locations exposed to extreme temperatures, humidity, direct sunlight, dust or corrosive vapours.
  - Locations exposed to vibrations, shocks or where it will be at an angle.

### Specifications:

Product description	OMRON Blood Pressure Monitor Cuff
Model	CS2 Small Cuff (HEM-CS24)
Arm circumference	17 to 22 cm
Cuff dimensions	Approx. 115 mm x 367 mm
Cuff pressure range	Pressure: 0 to 299 mmHg
Accuracy	Pressure: $\pm 3$ mmHg
Classification	Class I
Operating conditions	+10 to +40°C (50 to 104°F) 15 to 90% RH (non-condensing) 700 to 1060 hPa
Storage/Transport conditions	-20 to +60°C (-4 to 140°F) 10 to 95% RH (non-condensing) 700 to 1060 hPa
Package contents	Cuff, instruction manual, air plug
Durable period (Service life), Warranty	1 year

Please report to the manufacturer and the competent authority of the Member State in which you are established about any serious incident that has occurred in relation to this device.

### Symbols description

	CE Marking
	LOT number
	Medical device
	Temperature limitation
	Humidity limitation
	Atmospheric pressure limitation
	Identifier of cuffs compatible for the device
	Cuff positioning indicator for the left arm
	Range pointer and brachial artery alignment position
	Marker on the cuff to be positioned above the artery
<b>LATEX FREE</b>	Not made with natural rubber latex
	Range indicator of arm circumferences to help selection of the correct cuff size
	Reference catalogue number
	Easy Cuff
	Intelli Wrap Cuff
	Comfort Cuff
 TECHNOLOGY BY OMRON HEALTHCARE IN JAPAN	Technology by OMRON Healthcare in Japan
	Date of manufacture

Need for the user to consult the instruction manual

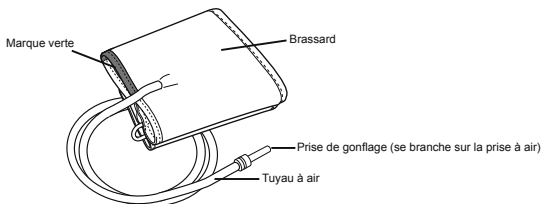
Arm circumference

Product production date is integrated in the Lot number, which is placed on the product and/or sales package: the first 4 digits mean year of production, the next 2 digits mean month of production and the last 2 digits mean day of production.

<b>Manufacturer</b> 	<b>OMRON HEALTHCARE Co., Ltd.</b> 53, Kunotsubo, Terado-cho, Muko, KYOTO, 617-0002 JAPAN
<b>EU-representative</b> <div style="border: 1px solid black; padding: 2px; display: inline-block;"> <b>EC REP</b> </div>	<b>OMRON HEALTHCARE EUROPE B.V.</b> Scorpius 33, 2132 LR Hoofddorp, THE NETHERLANDS <a href="http://www.omron-healthcare.com">www.omron-healthcare.com</a>
<b>Importer in EU</b>	
<b>Production facility</b>	<b>OMRON DALIAN Co., Ltd.</b> No. 3, Song Jiang Road, Economic and Technical Development Zone, Dalian 116600, China
<b>Subsidiaries</b>	<b>OMRON HEALTHCARE UK LTD.</b> Opal Drive, Fox Milne, Milton Keynes, MK15 0DG, UK <a href="http://www.omron-healthcare.com">www.omron-healthcare.com</a>
	<b>OMRON MEDIZINTECHNIK HANDELSGESELLSCHAFT mbH</b> Konrad-Zuse-Ring 28, 68163 Mannheim, GERMANY <a href="http://www.omron-healthcare.com">www.omron-healthcare.com</a>
	<b>OMRON SANTÉ FRANCE SAS</b> 3, Parvis de la Gare, 94130 Nogent-sur-Marne, FRANCE <a href="http://www.omron-healthcare.com">www.omron-healthcare.com</a>

Made in China

This OMRON product is produced under the strict quality system of OMRON Healthcare Co., Ltd., Japan.

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