DE EN FR IT

**Produktinformationen** / Product information  
Information sur le produit / Informazioni sul prodotto

# siegmund®

## FFP2 Atemschutzmaske / Particle filtering half mask Masque filtrant jetable / Facciale filtrante monouso

**CE 2163**  
EN 149:2001+A1:2009 FFP2 NR

<b>Produktname</b>	<b>FFP2 Atemschutzmaske</b>
Product name / Type de produit / Tipo di prodotto	FFP2 Particle filtering half mask / Masque filtrant jetable FFP2 / Facciale filtrante monouso FFP2
<b>Modell</b>	<b>Air Queen Breeze Mask</b>
Model / Modèle / Modello	
<b>Verordnung</b>	<b>(EU) 2016/425</b>
Regulation / Réglementation / Regolamento	
<b>Material</b>	<b>Vliesstoff + Filter (Nanofaser) + kunststoffbeschichteter Draht + Polypropylenring + Nylonband</b>
Material / Matériel / Materiali	non-woven + filter (nanofiber) + plastic coated wire + polypropylene ring + nylon strap / de non-tissé + filtre (nanofibre) + fil plastifié + anneau en polypropylène + sngle en nylon / tessuto non tessuto + filtro (nanofibra) + filo plastificato + anello in polipropilene + cinturino in nylon
<b>Gültigkeitszeitraum</b>	<b>3 Jahre nach Produktionsdatum, siehe innenliegendes Zertifikat</b>
Validity period / Durée de validité / Durata della validità	3 years after production date, see certificate inside / 3 ans après la date de production, voir certificat à l'intérieur / 3 anni dalla data di produzione, vedere il certificato all'interno
<b>Artikelnummer</b>	<b>4000</b>
Article number / Numéro de l'article / Numero dell'articolo	
<b>Prüfnorm</b>	<b>EN 149:2001+A1:2009 FFP2 NR</b>
Test standard / Norme de contrôle/ Standard di prova	
<b>PPE Ausstellende Behörde</b>	<b>UNIVERSAL CERTIFICATION and SURVEILLANCE SERVICES Trade Co.</b>
PPE issuing authority / Autorité de délivrance DPI / Autorità emittente DPI	
<b>Adresse der ausstellenden Behörde</b>	<b>Necip Fazil Bulvari Keyap Sitesi E2 Blok No:44/84 Yukari Dudullu Ümraniye/ISTANBUL - TURKEY</b>
Address of issuing authority / Adresse de l'autorité émettrice / Indirizzo dell'autorità emittente	
<b>Notifizierte Stelle</b>	<b>NB2163</b>
Notified Body / Organisme d'inspection notifié / Organismo di controllo notificato	
<b>Import in die EU &amp; Vertrieb</b>	<b>Siegmund Care GmbH</b>
Import to the EU & Sales / Importateur et distributeur dans l'UE / Importatore & Distributore nelle EU	Landsberger Straße 180, 86507 Oberottmarshausen, Deutschland / Germany / Allemagne / Germania, Telefon: (+49) 0 82 33 / 84 89 19 - 0, Internet: www.siegmund.care
<b>Hersteller</b>	<b>TOPTEC Co., Ltd.</b>
Manufacturer / Fabricant / Fabbicante	122, Asanvalley-ro, Dunpo-myeon, Asan-si, Chungcheongnam-do, Republic of Korea

DE EN FR IT

**Produktinformationen** / Product information  
Information sur le produit / Informazioni sul prodotto**siegmund**<sup>®</sup>**DE Warnhinweise**

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**EN Warnings**

1. Make sure not to place towel, tissue, etc. inside the mask.
2. Do not use if the inside of the mask is contaminated.
3. Do not wash for use.
4. Do not crush or deform the mask.
5. Do not touch the surface of the mask after wearing.
6. If you are a pregnant women, a child, a senior citizen, a patient suffering from respiratory tract or cardiovascular disease and feel uncomfortable in breathing when you wear the mask please discontinue using immediately and consult your doctor or other specialist if necessary.

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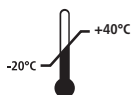
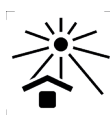
### EN Instruction Manual

1. Face the side of the mask without nose clip, then pull the ear loops with two hands to make sure that the nose clip is on the above.
2. Put mask below the chin.
3. After pulling the ear loop to the ear, adjust the ear loop to make yourself comfortable.
4. Place your fingers on the center of the metal nose clip and move your fingers to the sides along the nose clip accordingly until the nose clip is completely in the shape of your nose bridge. Caution! Adjusting the nosepiece with just one hand can cause the mask to leak.
5. Before entering the working area, the user must check if the mask is tightly fitting:
  - I) Cover the mask with two hands to fix the position of the mask on the face.
  - II) If air is leaking from the nose bridge, please readjust the nose clip according to step 4. If air is leaking from the edge of the mask, please readjust the ear loop. If the air leaking problem still cannot be solved, please repeat step 1-4.
  - III) If there is no leakage, you can continue to wear it.

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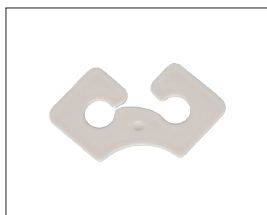
### EN Instruction Manual

1. Face the side of the mask without nose clip, then pull the ear loops with two hands to make sure that the nose clip is on the above.
2. Take the enclosed clip and hang it with the more closed side in one ear loop.
3. Put mask below the chin.
4. After you have put the ear loops around your ears, bring the two ear loops together at the lower back of the head and hang the ear loop on the free hook of the clip.
5. Place your fingers on the center of the metal nose clip and move your fingers to the sides along the nose clip accordingly until the nose clip is completely in the shape of your nose bridge. Caution! Adjusting the nosepiece with just one hand can cause the mask to leak.
6. Before entering the working area, the user must check if the mask is tightly fitting: I) Cover the mask with two hands to fix the position of the mask on the face. II) If air is leaking from the nose bridge, please readjust the nose clip according to step 4. If air is leaking from the edge of the mask, please readjust the ear loop. If the air leaking problem still cannot be solved, please repeat step 1-4. III) If there is no leakage, you can continue to wear it.

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